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1. Specifications and Scope of Supply

Item no.: PM-2010
Material: PES/Carbon, latex free
Colour: blue/red
ILN: 4250355320100

Item no.: PM-2020
Material: Cotton/Naturdrell, latex free
Colour: crème
ILN: 4250355320209

Dimensions: 50 x 60 cm
Storage: -5 – 40° C
 protect from sunlight

Scope of supply:

- Sliding mat
- Manual

Made in Germany

2. Use / Placement

The sliding mat makes it possible to not only move the patient within the bed but also transfer the patient to a wheelchair, toilet, chair or car seat. It is ideal for use in conjunction with the sliding board, turntable, raising aid or holding belt.

For positioning the patient or turning the patient onto their side the sliding mat is inserted transversely underneath the patient's pelvis. Now the patient can turn on their own or with trusted help.

This product is suitable for re-use.

3. Maintenance / Care

The product does not require any special maintenance. It can be washed in a washing machine at 90° C and is suitable for low dryer temperatures (45 – 50° C). When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the product can strongly degrade. The robust material can be disinfected with the usual wipe disinfectants. Check the compatibility before use where required on the basis of the disinfectant's instructions. We recommend the use of alcohol- and aldehyde-free agents, e.g. *Trionio®*. There is also some care information on the label.

4. Warranty

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.

5. Visual and Function Inspection / Safety Information / Re-use

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly and especially before re-use. Make sure that the product is in good condition and that the material and the seams are not damaged. Check the function of the product and make sure that the material resists the load. The smooth surface of the low-friction material makes it easy to slide and move. The best way to test these properties is by comparing the product with a new product.

For re-using the product it must be washed and disinfected before, see 3.

WARNING:

- ***Faulty products must no longer be used.***
- ***If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.***
- ***When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the product can strongly degrade.***
- ***The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the transfer surfaces, making sure they are stable and brakes are applied (e. g. wheelchair, bed).***

6. Product Labeling

The Petermann Alpha[®] sliding mat 50 x 60 cm is a medical product class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

7. Disposal Information

For the sliding mat there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

+49 (0) 9868 9339-0

Petermann GmbH • Ziegelhaus 12 • D-91601 Dombühl
Postal address: Petermann GmbH • Waldweg 3 • D-91601 Dombühl
Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20
E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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Petermann GmbH est certifié par le TUV conforme à la directive DIN EN ISO 13485.
Adresse postale: Petermann GmbH • Waldweg 3 • D-91601 Dombühl