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## 1. Specifications and Scope of Supply

**Item no.:** PM-6013 S  
**Dimensions:** 105 x 12 cm  
**Width:** 60 – 110 cm  
**ILN:** 4250355360137

**Item no.:** PM-6014 M  
**Dimensions:** 120 x 12 cm  
**Width:** 72 – 140 cm  
**ILN:** 4250355360144


**Item no.:** PM-6015 L  
**Dimensions:** 135 x 12 cm  
**Width:** 100 – 160 cm  
**ILN:** 4250355360151

**Material:** Nylon, latex free  
**Colour:** red  
**Storage:** -5 – 40° C  
 protect from sunlight


**Scope of supply:**

- Holding belt
- Manual

**Made in Germany**

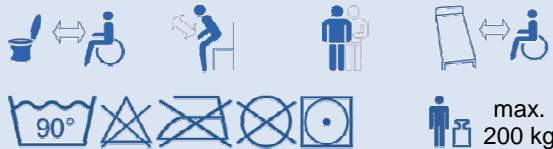
max. 200 kg
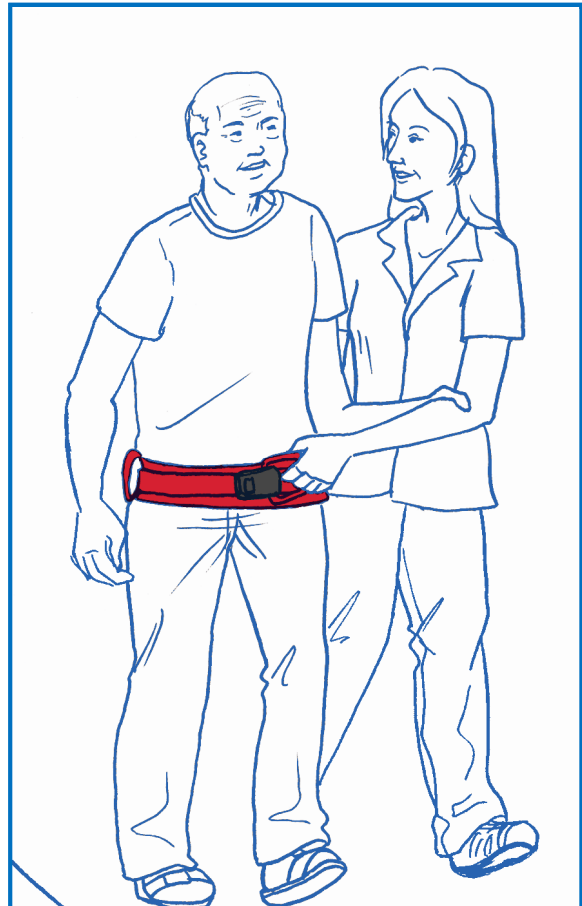
## 2. Use / Placement

The belt is designed to support a patient getting up and sitting down and when transferring a patient to a wheelchair, toilet, chair or car seat. The carer controls the flow of movement properly and corpulent patients or unstable and insecure patients can be held more securely. Anxious patients tend to feel safe. The carer can also wear a belt in order to provide the patient with a good, secure grip. The belt is placed round the patient's waist and closed with the aid of an adjustable quick-release clasp.

**This product is suitable for re-use.**

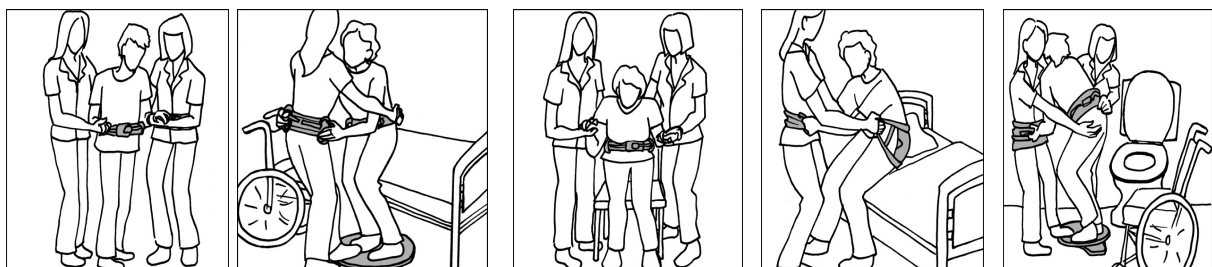### 3. Maintenance / Care

---

The product does not require any special maintenance. It can be washed in a washing machine at 90° C. When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the product can strongly degrade. The robust material can be disinfected with the usual wipe disinfectants. Check the compatibility before use where required on the basis of the disinfectant's instructions. We recommend the use of alcohol- and aldehyde-free agents, e.g. *Trionic®*. There is also some care information on the label.

### 4. Warranty

---

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.

### 5. Visual and Function Inspection / Safety Information / Re-use

---

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly and especially before re-use. Make sure that the product is in good condition and that the material and the seams are not damaged. Check the function of the product by applying pressure and loading to the handles and the clasp and make sure that the material resists the load. The high-friction has a rough surface to prevent slippage. The best way to test these properties is by comparing the product with a new product.

For re-using the product it must be washed and disinfected before, see 3.

#### **WARNING:**

- ***Faulty products must no longer be used.***
- ***If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.***
- ***When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the product can strongly degrade.***
- ***Carefully check the transfer surfaces, making sure they are stable and brakes are applied (e. g. wheelchair, bed).***

### 6. Product Labeling

---

The Petermann Alpha<sup>®</sup> holding belt is a medical product class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

### 7. Disposal Information

---

For the holding belt there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

**+49 (0) 9868 9339-0**

Petermann GmbH • Ziegelhaus 12 • D-91601 Dombühl  
Postal address: Petermann GmbH • Waldweg 3 • D-91601 Dombühl  
Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20  
E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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Petermann GmbH est certifié par le TUV conforme à la directive DIN EN ISO 13485.
Adresse postale: Petermann GmbH • Waldweg 3 • D-91601 Dombühl