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1. Specifications and Scope of Supply

Item no.:	PM-5010
Dimensions:	75 x 16 cm
ILN:	4250355350107
Item no.:	PM-5020
Dimensions:	58 x 32 cm
ILN:	4250355350206
Item no.:	PM-5050
Dimensions:	73 x 32 cm
ILN:	4250355350503
Material:	Polyethylene (PE), latex free
Colour:	black
Dimensions:	140 x 90 cm
Storage:	-5 – 40° C protect from sunlight

Scope of supply:

- Sliding board
- Manual

Made in Germany

Sliding board Quick (Item no. PM-5010)

2. Use / Placement

The sliding board is designed for transferring a patient, e. g. to a wheelchair, toilet or car seat. The patient no longer has to stand up but can make the change in position while seated, either with or without assistance.

The sliding board Quick is particularly small and handy and is intended for mobile use.

This product is suitable for re-use.

Examples of use:

3. Maintenance / Care

The sliding board does not require any special maintenance. You can use normal household cleaner and wipe it down with a moist cloth regularly for reasons of hygiene. A wipe-down disinfection can be performed with the usual disinfectant solutions and isopropyl alcohol. There is also some care information on the label.

4. Warranty

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.

5. Visual and Function Inspection / Safety Information

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly and especially before re-use. Make sure that the product is in good condition and that the material is not damaged. Check the function of the product and make sure that the material resists the load and does not tear. The smooth surface of the low-friction material makes it easy to slide and move the board. The high-friction has a rough surface to prevent slippage. The best way to test these properties is by comparing the product with a new product. For re-using the product it must be washed and disinfected before, see 3.

WARNING:

- **Faulty products must no longer be used.**
- **If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.**
- **Make sure the sliding board is stable on both sides of the transfer surfaces. The distance between free transfer surfaces using the sliding board must not be more than 20 cm (10 cm in the case of PM-5020, 5 cm in the case of PM-5010).**
- **Take note of the restricted sliding surface when using the Quick board.**
- **Transfers with bare skin must be supported with an intermediate layer (e. g. towel).**
- **The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the transfer surfaces, making sure they are stable and brakes are applied (e. g. wheelchair, bed).**

6. Product Labeling

The Petermann Alpha[®] sliding board is a medical product class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

7. Disposal Information

For the sliding board there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

+49 (0) 9868 9339-0

Petermann GmbH • Ziegelhaus 12 • D-91601 Dombühl
Postal address: Petermann GmbH • Waldweg 3 • D-91601 Dombühl
Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20
E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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Petermann GmbH est certifié par le TUV conforme à la directive DIN EN ISO 13485.
Adresse postale: Petermann GmbH • Waldweg 3 • D-91601 Dombühl