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(D)

(GB)

Seite 2

Page 5

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1. Specifications and Scope of Supply

Item no.:	PM-1500
Alpha transfer sheet 400 incl. PEM disposable sliding sheet (Item no. PM-060.60-100)	
ILN:	4250355315007
Item no.:	PM-1501
Alpha transfer sheet 400	
ILN:	4250355315014
Dimensions:	220 x 115 cm
Material sheet:	PES with PVC, latex free, water proof
Material grips:	Polypropylene (PP), latex free
Grips:	12 pieces (4 on each of the long sides, 2 at the head, 2 at the foot)
Colour:	white
Storage:	-5 – 40° C protect from sunlight

Specifications of the sliding foil see manual "PEM disposable sliding foil".


Scope of supply:

- Transfer sheet with grips
- PEM disposable sliding foil
- Manual

Made in Germany

max.
200 kg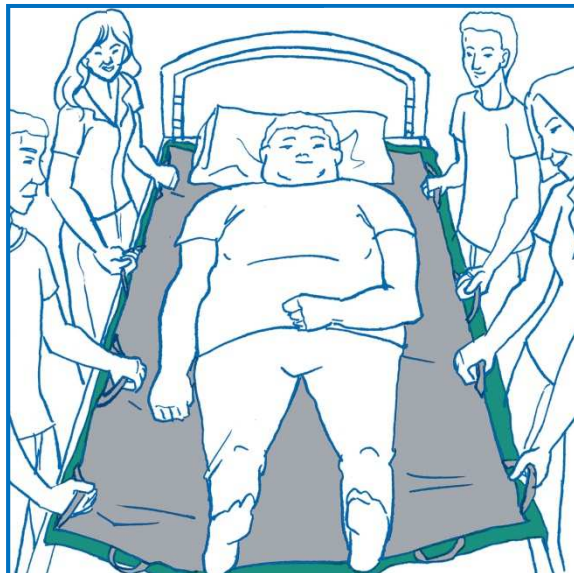

2. Use / Placement

The Alpha® transfer system 400 is a very simple design and very easy to use, which allows it to be deployed very quickly. It is designed for use in extreme transfer situations involving heavy patients. The support sheet can be safely lifted and carried by a maximum of 12 persons. Every carrier loop is attached to the opposite loop in the belt system.

This product is suitable for re-use.

The PEM disposable sliding foil is used to assist in moving patients on solid surfaces and bases. It is pulled from the roll to the required length, in several pieces if necessary.

WARNING:

- **It is very important to have a sufficient number of carers and assistants for transfer situations of patients in this weight class.**
- **Always take care of your back in such transfer situations.**
- **Distribute loads evenly and always be aware of your back when working.**

2.1 Example: Transfer from bed to a different level

WARNING:

- ***The transfer situation in this weight class always requires a new assessment of the situation and required actions depending on the place and persons involved.***
- ***These instructions can therefore only be regarded as a general guide for the actual requirements and the situation.***

Preparation: cut two lengths of 200 cm each. The more sheets laid side by side and under the person, even in different lengths, the easier it is to move the patient. Place the patient in a Sims position as much as possible. Fold the sheet (1) under one third and place it on the patient. Place sheet (2) beside sheet (1). Sheet (2) can be partially on top of sheet (1) depending on the space available. If necessary additional sheets can be used. Lock the transfer surfaces to prevent movement and slippage. Return the patient to a supine position. The sheets can also be placed under the bed sheets in the same way as described above. This allows the bed sheets to be used to assist in sliding in this situation. Now place the transfer sheet under the sheets and the patient from an accessible side. The transfer sheet can be placed between several layers of slide sheets if appropriate. Place the patient in a Sims position and place the transfer sheet folded one third under the sheets and the patient. Return the patient to a supine position to be able to pull the transfer sheet out to the other side. The transfer surfaces should be as close together as possible. Pull the transfer sheet with the patient on it to the other surface by placing slide sheets under the transfer aid. After the transfer remove the slide sheets and the transfer sheet.

3. Maintenance / Care

The transfer sheet does not require any special maintenance. You can use a normal household cleaner and wipe it down with a moist cloth (up to 70° C) regularly for reasons of hygiene. The transfer sheet can be disinfected with the usual wipe disinfectants. Check the compatibility before use where required on the basis of the disinfectant's instructions. We recommend the use of alcohol- and aldehyde-free agents, e.g. *Trionic®*. There is also some care information on the label.

4. Warranty

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.

5. Visual and Function Inspection / Safety Information / Re-use

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly and especially before re-use. Make sure that the product is in good condition and that the material and the seams are not damaged. Check the function of the product by applying pressure and loading to the handles and make sure that the material resists the load. The high-friction has a rough surface to prevent slippage. The best way to test these properties is by comparing the product with a new product.

For re-using the product it must be washed and disinfected before, see 3.

WARNING:

- ***Faulty products must no longer be used.***
- ***If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.***
- ***Sufficient carers and assistants are required for transfers to ensure complete safety during the transfer procedure.***
- ***Follow the load handling regulations when lifting patients.***

- *The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the transfer surfaces, making sure they are stable and brakes are applied (e. g. wheelchair, bed).*

6. Product Labeling

The Petermann Alpha® transfer sheet 400 is a medical product class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

7. Disposal Information

For the transfer sheet there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

+49 (0) 9868 9339-0

Petermann GmbH • Ziegelhaus 12 • D-91601 Dombühl
Postal address: Petermann GmbH • Waldweg 3 • D-91601 Dombühl
Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20
E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"